EN Instructions for use

Salvelox Foot Rescue
Intended use: Treatment of foot fungus and simultaneous treatment of heel cracks (rhagades), calluses (hyperkeratosis) and dry feet.

1. How does Salvelox Foot Rescue work?
 Salvelox Foot Rescue is a patented medical device class Ia that has water-binding and antimicrobial properties.

2. Ingredients
 Aqua, 1,5-Pentanediol, Cetyl Alcohol, Caprylic/Capric Triglyceride, Glycerin, Isostearyl Isostearate, Dimethicone, Glyceryl Stearate, PEG-75 Stearate, Ceteth-20, Steareth-20, Xanthan Gum, Artemisia Abrotanum Leaf/Stem Extract, Disodium Phosphate, Sodium Phosphate.

3. Use
 Salvelox Foot Rescue can be used to treat and prevent the spread and / or aggravation of foot fungus, heel cracks, calluses and dry feet. To be used by adults. Apply 1 pump on each foot. The volume depends on the size of the treatment area. Apply to clean and dry feet so that the entire area to be treated is covered. Treat both feet to prevent spreading. Wash hands after application to avoid contamination of other areas.

4. Treatment
 Apply twice a day until all symptoms are completely eliminated. The treatment time is individual and depends on the initial severity of the symptoms. Initial effect on symptoms is usually seen after two weeks of treatment. For treatment of foot fungus an eight-week treatment period is recommended. It is important not to finish the treatment too early to prevent recurrence. If no improvement can be seen after 4 weeks, consider alternative treatment and/or contact a medical professional for advice. If you are a diabetic, suffer from reduced blood circulation or recurrent and / or long-term problems with your feet contact a medical professional for advice.

5. Description of the container
 Salvelox Foot Rescue is packed in an airtight bottle that protects the product from external influences. Thanks to the airtight package, a long shelf life is obtained when the product is protected from the environment. The bottle delivers about 1 ml per pump.

6. Warning and precautions
 • To be used by adults. Salvelox Foot Rescue is intended only for use on the feet.
 • Do not use on open wounds or on other parts of the body. Do not use the product if the package is damaged or if the expiry date stated on the package has passed. In case of accidental ingestion, a health care professional should be consulted, and this leaflet provided. Do not use if you are allergic or hypersensitive to any of the ingredients included or if you suffer from glutamic acid or kidney insufficiency. Do not combine Salvelox Foot Rescue with other products intended for foot fungus without first consulting a health care professional. Discontinue use of the product if irritation or allergic reaction occurs.
 • In case of any side effects or any abnormal reaction, a health care professional should immediately be consulted, and this leaflet provided. Consult a health care provider if the symptoms

persist after 4 weeks or if they get worse. Before use, consult your doctor if you are diabetic, suffer from reduced blood circulation or recurrent and / or long-term problems with your feet.

7. Shelf life and storage
 The shelf life is stated with an expiration date on the packaging. Keep out of the reach and sight of children and at room temperature.

8. Contact information
 Distribuidor: Orkla Care AB (Svetsvärgen 15) Box 1336, SE-171 26 Solna, Sweden. For distributors' contact details, please visit www.salvelox.com. Manufacturer: Natumin Pharma AB, Snickaregatan 10, 566 33 Habo, Sweden. Latest update: April 2020.

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