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Orkla Wound Care AB (Svetsarvägen 15),
Box 1336, SE-471 26 Solna, Sweden

For distributors' contact details,
please visit
www.salvelox.com

EN Blister Prevention protects the skin from friction and rubbing, helping to prevent blisters on the feet. Made from hydrocolloid material, it provides instant relief from pressure and friction while keeping the skin protected. It protects blisters and promotes faster healing by maintaining a moist environment, which research shows is beneficial for wound healing. The plaster has a discreet and transparent design, is waterproof, protects against dirt, and stays in place.

User instructions:

1. The skin should be clean and dry, free from any dirt or lotion.
2. Remove the protective plastic layer without touching the adhesive.
3. Apply the plaster and then remove the protective paper.
4. Attach the plaster and ensure the edges are secure by smoothing it onto the skin.

Keep the plaster on until it falls off naturally, not more than 5 days. Grabbing one end and stretching carefully along the skin easily removes the plaster.

PRECAUTIONS:

- People with impaired wound healing, e.g. diabetics or elderly people, should check blisters/wounds regularly.
- Contraindication: Do not use on blood blisters or deep heavily bleeding or infected blisters.
- Consult a doctor if an infection occurs or if a blister/wound has not healed within a week.
- For single use only. Do not reuse. Reusing the product increases the risk of infection.

• In the case of any serious incident associated with the use of this product please report it to the manufacturer.

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