Orkla Natural Healing Dressing (NORSK: Antibakteriell bandasje)

ENGLISH

Can be used for most types of wounds, including infected wounds and first and second degree burns (exception: third degree burns).

Medical grade honey provides natural antibacterial properties.

Promotes the wound healing process with its hydrogel wound pad, naturally antibacterial medical grade honey and accelerating wound healing, preventing infections and reducing the risk of scarring.

It has a cooling effect, soothes and calms the wound.

Transparent, discreet, soft and flexible design.

Highly absorbent: Absorbs exudate 7 times its weight.

With adhesive edges: No need for secondary bandage or fixation.

Does not contain any drugs.

How to use it:

Cleanse the wound as needed before applying the dressing. Take the sterile product out of the pouch. Remove backing and press firmly to apply the dressing to the wound. Check the wound and the surrounding tissue regularly.

One dressing can be worn for 3-5 days without changing.

If the wound is very moist or if the dressing has been damaged, it may need to be replaced more frequently.

Caution:

Consult your physician if signs of infection occur (redness, swelling, high fever, etc.). This product contains 30% honey:

Do not use if you are allergic to honey.

Do not use if the primary package is damaged or open.

Single-use product, do not re-use. The re-use of the product may cause infection.

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