Orkla Blister Natural Healing (NORSK: Antibakterielt Gnagsårplaster

ENGLISH

EN Blister Natural Healing is designed for severe blisters on the feet, including punctured, hard-to-heal, and infected blisters. It features a hydrogel wound pad infused with naturally antibacterial, medical-grade honey, which helps create an optimal healing environment while reducing the risk of infection. The medical-grade honey promotes faster and more effective healing, while the hydrogel provides a cooling effect that soothes and calms the wound. The plaster cushions and protects the blister, preventing further damage while offering long-lasting comfort. Its discreet, transparent design is soft and flexible, allowing it to stay securely in place for 3–5 days without needing to be changed. Waterproof and breathable, it also shields against dirt. Additionally, the highly absorbent wound pad can retain exudate up to seven times its weight, ensuring a clean and effective healing process.

User instructions:

The skin should be completely clean, dry and free from any lotion. Take the sterile product out of the pouch. Remove backing and press firmly to apply the plaster to the wound. Blister Natural Healing is intended for use by consumers under appropriate guidance, with regular checks of the wound and surrounding tissue. One dressing can be worn for 3-5 days without changing. If the wound is very moist or if the dressing has been damaged, it may need to be replaced more frequently. No special training is required to use the product correctly, but users should carefully follow the instructions for use. Dispose of the used product as per local protocol.

Precautions:

Consult your physician if signs of severe infection occur (redness, swelling, high fever, etc). This product contains honey. Do not use in case of an allergy to beehive products or to any of the other components. Do not use if the primary package is damaged or open. The product should only be used once by a single user. Reusing the product increases the risk of infection. Do not re-sterilise. In case of a serious incident related to the device, please notify Theo Manufacturing B.V., Orkla Wound Care AB, or your local distributor.

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Distributed by Orkla Wound Care AB (Svetsvägen 15), Box 1336, SE-171-26 Solna, Sweden
Theo Manufacturing B.V., Sleperweg 44, 6222 NK Maastricht, The Netherlands